

# Kingfisher Pool Access Seat Battery Care



The battery is okay, no need for charging.



Battery needs charging.



Battery needs charging. Buzzer sound is provided when a button is pressed at this battery stage.



**How to use the lowering/lifting function on the control box**

### **(EMERGENCY USE)**

- On the control box there are either buttons or holes.
- The lifting arm can be lowered or lifted via pressing a pen (or similar) in the hole.
- This is a permitted method of lowering/lifting.
- The lowering/lifting "buttons" works as normal hand control buttons (you do not get extended functionality by using these when the battery is low).



**ParaMobility**  
Making everyday living easier

**Phone 1300 444 600**

Delivery, installation + servicing Australia wide  
enquiries@paramobility.com.au  
www.paramobility.com.au