

# Kingfisher Pool Access Seat User Guide



## INSTALLATION AT POOL DECK

- Wheel the unit to the pool deck and place the base plate over the anchor points. Ensure the wheels of the hoist are placed facing away from the pool.
- Insert the Eye Bolts into the holes and hand tighten only.

## USE OF THE LIFTER

- Always test the lifter by pressing up and down on the hand control before commencing a lift.
- Check safe working load (SWL) of lifter before attempting a lift and make sure the weight of person being lifted complies.
- Test the hand control buttons and monitor the movement of the hoist prior to use to ensure everything is working properly
- The arms of the hoist are weight bearing so the client can transfer easily.
- If you obtained a footrest, lift the seat and remove the wing nut from tubing. Place the footrest into the tubing and hand tighten the wing nut prior to use.

## GETTING STARTED

- **Install the battery** – The Battery should always be removed and replaced into the charger after use. Do not remove the hand control from the lifter as this will disrupt the control box mechanisms.
- **Ensure the Emergency STOP button is in the out position.** If this is pushed in the hoist will not operate.



Emergency  
STOP Button

Ensure a travel  
strap is in  
place before  
transporting



**ParaMobility**

Making everyday living easier

**Phone 1300 444 600**

Delivery, installation + servicing Australia wide  
enquiries@paramobility.com.au  
www.paramobility.com.au

**Never push on the actuator to rotate the lifting arm. It will damage the lifting mechanism.**



## REMOVING THE HOIST

- Wipe down the Kingfisher with clean water (not chlorinated) prior to packing away. **DO NOT WET CONTROL BOX.**
- Remove the battery and place it on charge.
- Bring the lifting arm up to a vertical position and fold up the seat and arms.
- Remove the eye bolts from the base plate and place them in a safe place ready for next use.
- Put the cover on your hoist.
- Stand behind the hoist and lean back onto the trolley. Secure a travel strap on the trolley before moving.

## BATTERY CARE

- After use, the battery should be placed in the charger to ensure it is fully charged when required. Battery cover should remain on the battery housing to avoid damage and corrosion.

## HAND CONTROL OVERRIDE FOR LOWERING OR LIFTING

- If hand control is damaged – using a pen tip (or similar), push in the holes as indicated on the control box as shown. Refer to the Battery Care Guide.



**Cover your Kingfisher Pool Hoist when not in use**