## Platypus Pool Steps Maintenance Guide





The Platypus Pool Steps require minimal maintenance.

Para Mobility only use 316 grade Stainless Steel to manufacture our products, due to the corrosion or oxidation (rust) resistant properties.

Tea Staining is a discolouration of the surface of the stainless steel, and occurs in higher temperatures and humid conditions. Whilst having the same look as rust, Tea Staining does not affect the structural integrity or the longevity of the material. Visually it is a discolouration of the surface, which tends to follow the "grain" of the surface finish.

To ensure your pool steps continue to look great and avoid tea staining we recommend washing the area regularly using clean fresh water and mild detergent using a cloth or soft brush.

After washing rinse off with clean water and wipe the surface dry with a soft absorbent cloth.

Do not use abrasive cleaners or a scouring pad of any type as they may cause scratching, damage, excessive wear and dull the finish.

Do not use waxes or furniture polish.





## Phone 1300 444 600