Platypus Pool Steps User Guide




CORRECT





ENTERING THE POOL

- The Platypus Pool Steps have been designed to be used as a ladder. It is critical that all users are instructed to enter and exit the pool facing the same direction – away from the pool.
- Never allow the user to enter or exit the pool facing forwards. Serious injury may occur.

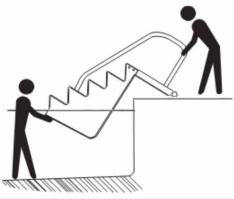

REMOVING THE STEPS

- Removing the steps is always a two-person job.
- Attach the dolly wheels to the standing platform before moving out of the water.
- Please use caution and care when removing the steps from the pool so as not to injure yourself.





Phone 1300 444 600 Delivery, installation + servicing Australia wide enquiries@paramobility.com.au www.paramobility.com.au

TRANSPORTING THE STEPS

- Never drag the steps across the pool deck.
- Attach the Dolly Wheels to the standing platform and hold underneath the bottom step when moving the steps.



CORRECT



NOT CORRECT







