

Pelican Pool & Spa Hoist Floating Sling Instructions

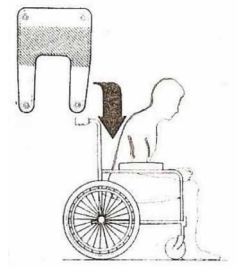
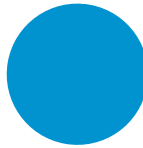


Figure 1

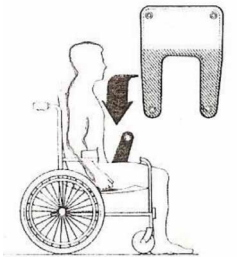


Figure 2

ENTERING THE WATER

1. Using the hand control, raise the Pelican lifter to well above the clients head, wheel the chair under the lifting hanger
2. Slide the sling down behind the back of the client (the grab handle should be at the top, facing you) and tuck the bottom of the sling as low as possible underneath the buttock area (figure 1).
3. Bring the legs of the sling from the outside of the thigh underneath and up between the legs ensuring you tuck the sling leg as high as possible underneath the thigh. Repeat the procedure on the other leg (figure 2)
4. Hook up the leg loops of the sling to the lifting hanger right to left, left to right.
5. Hook up the shoulder loops right to right, left to left. There should now be a total of four loops connected to the lifting hanger
6. Using the hand control, slowly raise the client off their chair and gently rotate the client over the water for lowering. A carer should be in the pool to accept the client if required.
7. Once the client is in the water, release the legs by unhooking the leg sections of the sling from the lifting hanger.
8. The client floats in the sling or floats away with the carer.

EXITING FROM THE WATER

1. Lower the sling to its maximum distance to allow easy access to the sling in the water
2. When the client is seated and the straps are connected correctly, raise the sling using the hand control above the pool deck and pull the arm of the lifter into the safe landing area and lower the client onto their chair for exiting.
3. When finished, lower the arm to its lowest position and install the travel pin to lock any movement.
4. Remove the battery and replace into charger.

WARNING

- Ensure that there are no obstacles in the path of the client being lifted.
- Check that arms and legs are clear of obstructions. Failure to do so may cause injury.



Para Mobility

Making everyday living easier

Phone 1300 444 600

Pelican® Installation & Service
Delivery, installation + servicing Australia wide
enquiries@paramobility.com.au
www.paramobility.com.au