

Pelican® Installation & Service Pool & Spa Hoist Maintenance Guide



The Pelican Pool and Spa Hoist has been manufactured in Australia and built to Australian Standards. It is important to regularly check over your pool hoist and associated products from Para Mobility on a regular basis.

The Pelican Pool and Spa Hoist is manufactured from 316 stainless steel. 316 stainless steel is a corrosion resistant alloy; however, it may display rusting characteristics under certain conditions such as in plant rooms or if the hoist is used in an indoor environment.

Following are some guidelines to assist you in developing a maintenance schedule for your hoist.

- Routinely wipe down all surfaces of the hoist, using fresh water (not chlorinated). This will minimise the potential build-up of corrosion or rust like symptoms such as tea staining.
- In the event the equipment displays stubborn stains or heavy deposit build up, specialised cleaning materials may be needed to remove foreign substances and restore the original finish.
- Always use the supplied cover when not in use.
- If you have a sling or aquatic wheelchair, regularly check the straps and stitching are not frayed or damaged. Ensure the SWL sticker remains on the chair. Check for any damage to the fabric or the frame of the chair.
- It is imperative that servicing is carried out a least annually by Para Mobility Pelican® Installation & Service or one of its qualified service agents.



Para Mobility

Making everyday living easier

Phone 1300 444 600

Pelican® Installation & Service
Delivery, installation + servicing Australia wide
enquiries@paramobility.com.au
www.paramobility.com.au



HAND CONTROL

Don't submerge handpiece in water for an extended period. **NEVER** take the hand control out of the control box at the bottom of the hoist unless instructed by Pelican® Installation & Service or your distributor. Check the hand control by regularly pressing the button in the up and down motion and before a lift to ensure it is working correctly. Dropping the hand control on the ground, could damage the internal workings, please take care at all times.



Safe Work Load (SWL)

Floating Hand Control

Actuator

Battery & Control Box



ACTUATOR

The actuator lifts the main arm of the hoist. The hoist must never be rotated by pushing on the actuator, this will damage the lifting mechanisms. Check to ensure it remains plugged into the bottom of the control box. It must never be removed from the control box unless instructed by Pelican® Installation & Service or your distributor.

When storing you pool hoist, **DO NOT** store it in a plant room where the hoist is exposed to harsh chemicals, these chemicals could damage some of the inner working of the hoist. The arm of the hoist must be kept down when not in use to minimise chemical build up on the actuator.

SAFE WORK LOAD STICKER

This sticker must remain on the hoist at all times. This is the weight the hoist has been tested to lift. If this needs replacing contact Para Mobility Pelican® Installation & Service.



Pelican Pool & Spa Hoist Cover Install Guide



1. Ensure locking pin is in.



2. Lower arm until lifting hanger just touches the ground.



3. Twist lifting hanger whilst lowering arm.



4. Pull cover over lifting hanger.



5. Pull cover over wheels.



Para Mobility

Making everyday living easier

Phone 1300 444 600

Pelican® Installation & Service
Delivery, installation + servicing Australia wide
enquiries@paramobility.com.au
www.paramobility.com.au