Pelican Pool & Spa Hoist Wheelchair Instructions





ENTERING THE WATER

- Place the client into the Pelican Wheelchair in the change room or desired location and wheel the chair to the pool hoist.
- 2. Unlock the lifter travel pin on the pool hoist.
- 3. Using the hand control, raise the hoist to above the client's head and wheel the chair under the lifting hanger.
- 4. Attach the chair straps to the lifting hanger.
- Using the hand control slowly raise the chair off the pool deck and gently swing it over the water for lowering. A carer should be in the pool to accept the person if required.
- 6. The client floats or walks off the chair.
- 7. Send the chair back to the pool deck if required.



EXITING FROM THE WATER

- Lower the chair to its maximum distance to allow easy access to the chair in the water
- When the client is seated correctly, raise the chair to slightly above the pool deck and pull the arm into the safe landing area and lower just enough to land the chair for alighting, then disconnect the straps. Wheel the chair to the change room.
- 3. When finished, lower the arm on the pool hoist to its lowest position and install the travel pin to lock the movement. Remove the battery to the charger.

NOTE

• Always lift the wheelchair arms in a vertical motion (as pictured).



Phone 1300 444 600

Pelican® Installation & Service Delivery, installation + servicing Australia wide enquiries@paramobility.com.au www.paramobility.com.au