



Para Mobility

Making everyday living easier

Phone 1300 444 600

Unit 31/276 New Line Rd

Dural NSW 2158 Australia

enquiries@paramobility.com.au

www.paramobility.com.au


Pelican Pool & Spa Hoist Measuring Guide

Customer Name _____ Phone _____

Pool Name _____

Install Address _____

Pelican Rim Setback (Aerial View)

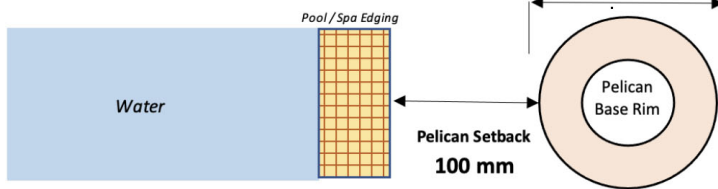

Pelican setback is the standard setback distance (100mm).

IMPORTANT NOTICE: WATER DEPTH

In accordance with the Disabilities Standards Code, Disability (Access to Premises – Buildings) Standards 2010 Part D5, the Pelican base rim must be positioned on the pool or spa surrounds where the **water depth is not less than 500mm and not more than 1300mm.**

Water depth at the point of entry (ie Pelican location) _____ mm (max 1300mm)

Which profile best matches your pool? ☐ A ☐ B ☐ C ☐ D ☐ E

Please include measurements in the profile you selected (see page 2).

Do you have the minimum concrete width & thickness to mount the hoist on pool deck? ☐ Y ☐ N

TO SAFELY INSTALL the Pelican Pool & Spa Hoist you will need a FLAT AND LEVEL SOLID CONCRETE PAD to anchor it to. We recommend a minimum $\frac{1}{4}$ of cubic metre of concrete is used when pouring your pad. This can be made up as a logical size, for example: 1000mm x 1000mm x 300mm but at a minimum must have a surface area of at least 650mm x 650mm & minimum depth of 100mm. The ideal location is placed up against the pool edge.

Which Dulux colour would you like?



☐ Pearl White Gloss

☐ Charcoal Metallic Pearl Gloss

☐ Silver Kinetic Pearl Satin

☐ Safety Yellow Gloss

☐ French Blue Gloss

☐ Primrose Gloss

What is the base substrate material the hoist will be installed on?

☐ Concrete Pad ☐ Tiles over Concrete ☐ Pavers over Concrete ☐ Timber Decking ☐ Other: _____

Property access considerations/risks? _____



ParaMobility

Making everyday living easier

Phone 1300 444 600

Unit 31/276 New Line Rd

Dural NSW 2158 Australia


enquiries@paramobility.com.au

www.paramobility.com.au

Pelican Pool & Spa Hoist Measuring Guide

ROUND MEASUREMENTS TO THE NEAREST 10MM

Profile A

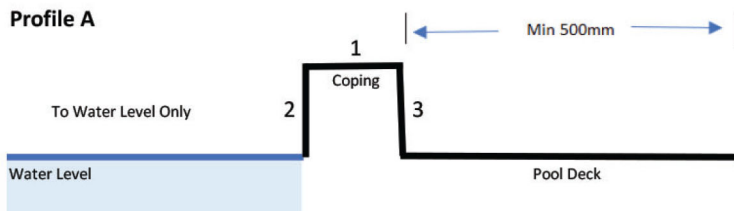


A1 Coping Width

A2 Internal Riser

A3 External Riser


NOTE: Measurement 2 may be different to measurement 3.

Profile B

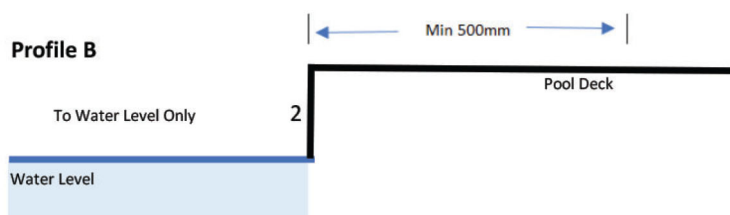

B2 Internal Riser

Profile C

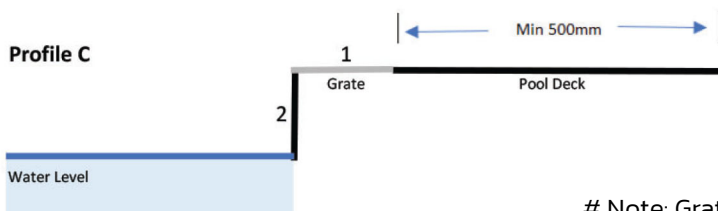


C1 Grate Width #

C2 Internal Riser

Note: Grate width is from pool edge to commencement of concourse.


Profile D

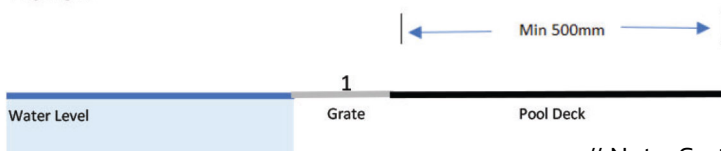

D1 Grate Width #

Note: Grate width is from pool edge to commencement of concourse.

Profile E

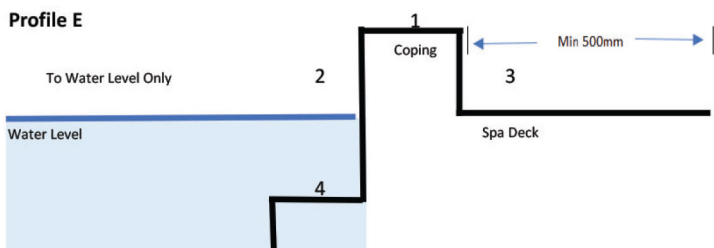

E1 Coping Width

E2 Internal Riser

E3 External Riser

E4 Width of
Spa Seat

NOTE: Measurement 2 may be different to measurement 3.