

Horizon Montar Wall Mounted Change Table - SWL 200kg

EXPLANATION OF FEATURES



Space Saving Position
The Montar in its space friendly position. When it's not in use, it can easily fold to save space and avoid trip hazards.



Unfolding Bed Base
LOCK - to fold the Montar's bed base to its lying position, put one hand on the top of the bed base - unlock - and slowly lower.



Handset Functions
Handset showing up/down buttons for raising or lowering the padded surface.



Patient Entering Position
The Montar in its patient entering position - this is the lowest position with the rail tucked underneath for easy access.
WARNING: No person should be left on the change table unattended.



Moving Rails to Fixed Position
Engaging rail to its upward fixed position by simply pulling the rail towards you as well as upwards. This will lock automatically once the rail is 90 degrees to the bed base.



Lowest Position
The Montar positioned in its lowest position with rail in its lowest position.
WARNING: Ensure safety rail is secured whenever a patient is lying on the surface.



Unlocking Rail
To unlock the rail, pull the lock knob, gently lower and push the rail back underneath the bed base to its home position.



Highest Position
The Montar in its highest position.
WARNING: DO NOT enter or leave the Montar whilst it's in the highest position.

MONTHLY MAINTENANCE - Parts to check on the Montar



Wear and Tear
Look for wear and tear on the walls, cracking cornices and listen for creaking (with weight on the Montar).



Structural Check
Ensure the Montar is fastened securely to the wall and fasteners are tight. Ensure there are no cracks in the walls or framework of the Montar.



Safety Rail Check
Ensure the safety rail bolts are secure and the rail is functioning properly.



Bed Base Check
Check the bed base hinge bolts are secure and tight.



Gas Strut Check
Ensure gas strut is secure, tight and functioning properly.



Feet Check
Make sure the feet are firm and there is weight onto the ground. If the feet are loose, they will need re-screwing downwards to force weight back to the ground.



Moving Parts Check
Spray moving parts and locks with some lubricant spray - this will enhance longer life to wearing parts.



Oil Guide Check
Spray and oil guide tubes through the cutouts on the wall unit cover.

MONTHLY CHECKLIST - Please use the below checklist when completing your monthly checks.

Record of Performed Service Procedures and Regular Service Maintenance												
YEAR:	JAN	FEB	MAR	APR	MAY	JUN	JULY	AUG	SEPT	OCT	NOV	DEC
Wear & Tear												
Structural												
Safety Rail												
Bed Base												
Gas Strut												
Feet												
Moving Parts												
Oil Check												

CLEANING THE MONTAR WALL MOUNTED CHANGE TABLE

- Cleaning can be done often - using domestic detergents.
- Do not use acids, alkaline or any other chemical solvents.
- When cleaning, be careful of entrapment, as it could cause physical injury.

CLEANING CONTACT:

MAINTENANCE CONTACT:

