

Horizon Montar Splash Wall Mounted Change Table - SWL 200kg

EXPLANATION OF FEATURES

| | | | | | | | |
|---|---|---|--|--|---|--|--|
|  |  |  |  |  |  |  |  |
| <p>Space Saving Position The Montar in its space friendly position. When it's not in use, it can easily fold to save space and avoid trip hazards.</p> | <p>Unfolding Bed Base LOCK - to fold the Montar's bed base to its lying position, put one hand on the top of the bed base - unlock - and slowly lower.</p> | <p>Handset Functions Handset showing up/down buttons for raising or lowering the padded surface.</p> | <p>Patient Entering Position The Montar in it's patient entering position - this is the lowest position with the rail tucked underneath for easy access. WARNING: No person should be left on the change table unattended.</p> | <p>Moving Rails to Fixed Position Engaging rail to its upward fixed position by simply pulling the rail towards you as well as upwards. This will lock automatically once the rail is 90 degrees to the bed base.</p> | <p>Lowest Position The Montar positioned in it's lowest position with rail in its lowest position. WARNING: Ensure safety rail is secured whenever a patient is lying on the surface.</p> | <p>Unlocking Rail To unlock the rail, pull the lock knob, gently lower and push the rail back underneath the bed base to its home position.</p> | <p>Highest Position The Montar in its highest position. WARNING: DO NOT enter or leave the Montar whilst it's in the highest position.</p> |

MONTHLY MAINTENANCE - Parts to check on the Montar Splash

| | | | | | | | |
|---|--|---|---|---|---|---|---|
|  |  |  |  |  |  |  |  |
| <p>Wear and Tear Look for wear and tear on the walls, cracking cornices and listen for creaking (with weight on the Montar).</p> | <p>Structural Check Ensure the Montar is fastened securely to the wall and fasteners are tight. Ensure there are no cracks in the walls or framework of the Montar.</p> | <p>Safety Rail Check Ensure the safety rail bolts are secure and the rail is functioning properly.</p> | <p>Bed Base Check Check the bed base hinge bolts are secure and tight.</p> | <p>Gas Strut Check Ensure gas strut is secure, tight and functioning properly.</p> | <p>Feet Check Make sure the feet are firm and there is weight onto the ground. If the feet are loose, they will need re-screwing downwards to force weight back to the ground.</p> | <p>Moving Parts Check Spray moving parts and locks with some lubricant spray - this will enhance longer life to wearing parts.</p> | <p>Oil Guide Check Spray and oil guide tubes through the cutouts on the wall unit cover.</p> |

MONTHLY CHECKLIST - Please use the below checklist when completing your monthly checks.

| Record of Performed Service Procedures and Regular Service Maintenance | | | | | | | | | | | | |
|--|-----|-----|-----|-----|-----|-----|------|-----|------|-----|-----|-----|
| YEAR: | JAN | FEB | MAR | APR | MAY | JUN | JULY | AUG | SEPT | OCT | NOV | DEC |
| Wear & Tear | | | | | | | | | | | | |
| Structural | | | | | | | | | | | | |
| Safety Rail | | | | | | | | | | | | |
| Bed Base | | | | | | | | | | | | |
| Gas Strut | | | | | | | | | | | | |
| Feet | | | | | | | | | | | | |
| Moving Parts | | | | | | | | | | | | |
| Oil Check | | | | | | | | | | | | |

CLEANING THE MONTAR SPLASH WALL MOUNTED CHANGE TABLE

- Cleaning can be done often - using domestic detergents.
- Do not use acids, alkaline or any other chemical solvents.
- When cleaning, be careful of entrapment, as it could cause physical injury.

CLEANING CONTACT:

MAINTENANCE CONTACT:

